

THE T-RING™

A Revolution in Finger First Aid



96113
(Includes 3
Finger Tourniquets)

The T-RING™ Finger Tourniquet is the only one-size-fits-all device that instantly stops bleeding from finger and toe cuts safely, painlessly, and effectively. Its patented, award-winning design automatically applies the correct amount of pressure, preventing over-tightening and ensuring comfort for every user.


Features:

- ✓ **Instant Bleeding Control:** Stops bleeding immediately upon application. No more fumbling with gauze or direct pressure
- ✓ **One Size Fits All:** Automatically adjusts to any finger or toe size
- ✓ **Safe & Painless:** Eliminates the need for painful direct pressure
- ✓ **Reduces Blood Exposure:** Minimizes risk of cross-contamination for both patient and responder
- ✓ **Proven & Trusted:** Used by medical professionals in 40 countries, with over 1 million uses
- ✓ **Compact & Affordable:** Small enough to fit in any first aid kit

Benefits:

- ✓ **Adds measurable value** to every first aid kit
- ✓ **Helps reduce OSHA recordables** and insurance mod hits
- ✓ **Prevents costly ER visits** (average bill > \$1,000)
- ✓ **Reduces lost work time** from minor injuries
- ✓ **Recognized by major organizations** including U.S. Armed Forces

How to Use the T-RING™ - 3 Easy Steps:

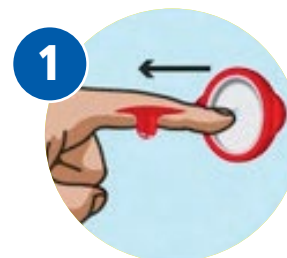


CONTROL

- Slide T-RING onto Finger
- Bleeding Instantly Stops!
- One Size Fits ALL!



CLEAN

- Wipe Clean
- Rinse Thoroughly
- Dry with Gauze

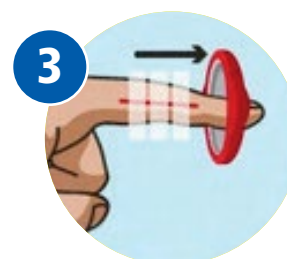

CLOSE

- Close with Wound Strips
- **Remove T-RING!** (max 60 min)
- Apply Bandage